

## IMPORTANT NOTICE

### CORRECTIVE ANNEX REGARDING CHARGING

Use **ONLY** the instructions in this notice to charge the batteries of your wheelchair. The original text in the instruction manual with regard to charging the batteries of the wheelchair, will lapse.

This annex is part of the instruction manual of the wheelchair.

We strongly advise you to read this notice carefully. If you have any questions, do not hesitate to contact your specialist dealer. He/she will be pleased to help you with your question.



## Safety instructions

 **WARNING** Risk of injuries and damage due to fire

- Only use the battery/batteries and charger supplied with your product. The use of any other battery or charger may be dangerous (fire hazard).  
Contact your specialist dealer for possible replacement.
- Under no circumstances open or change the battery, charger, cables, plugs nor change any connection points yourself!
- The charger is only intended to charge the battery/batteries delivered with the wheelchair, not to charge any other batteries.
- Protect the battery and battery charger against flames, high temperatures (>50°C), moisture, sunshine, severe shocks (for example dropping). Do NOT use the battery if this has been the case.
- Charge the battery with the supplied charger, indoors, in a well-ventilated area, out of children's reach.
- Do not use any extension cord for charging.

 **CAUTION** Risk of injuries

The batteries will last shorter or longer depending on the use, terrain and driving conditions.

When you plan a trip, take the battery-charge status in to account. Be aware of the action radius of the battery, see the technical details in the instruction manual.

**CAUTION**

Risk of battery damage and/or reduction of battery life span

- Read the charger instruction manual and observe the safety guidelines regarding the charger.
- Do not interrupt the charging cycle; wait until the charger indicates that the battery is fully charged, see the charger instruction manual. Prevent charging in short periods. Note: this is NOT applicable for Lithium batteries.
- The battery may be irreversibly damaged if the battery is fully discharged. Therefore, make sure that the battery is charged on time:
  - With intensive use (long distances, daily use): Immediately recharge after use. Do not interrupt the charging cycle.
  - With average use (short distances, daily or a few times a week): Charge when battery indicator shows 50% discharge. Do not interrupt the charging cycle.
  - With little use or storage: Charge once a week. Do not interrupt the charging cycle.
- The lifespan reduces if the battery is exposed to extreme cold for a longer period of time, see the technical details in the instruction manual.
- Make sure that all connection points on the battery and the connection point in the battery housing are free of dust and other contamination.
- Protect your battery from large sources of electromagnetic radiation.



## Charging instruction

1. Switch the wheelchair OFF.
2. FIRST connect the battery charger to a 230V AC power socket.
3. Connect the charger to the battery.
4. Refer to the battery-charger manual for the explication of the indicators on the charger.
5. When the battery is fully charged, disconnect the battery from the charger.
6. Remove the charger from the socket, let it cool down and store in a dry location.
7. Switch the wheelchair ON and check if all the lights on the battery status indicator are ON.

**i** In case of any problems whereby the battery cannot be charged according to the instructions in the charger manual, please contact your specialist dealer.

When the operating console displays the error code “Drive inhibit” after charging, make sure that the battery charger is disconnected. This safety measure prevents you from driving with the charger still connected to the supply grid.

