# VERMEIREN

## Streeter

USER MANUAL  
MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

EN

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## Content

Content .....	1
1 Preface .....	1
2 Your product .....	1
3 Before use .....	1
4 Assembly and adjustments .....	2
5 Using your Streeter .....	2
6 Maintenance .....	3
7 Technical specifications .....	3

## 1 Preface

Congratulations! You are now owner of a Vermeiren Streeter!

Thank you for your trust in the products of Vermeiren. To support you on the use of this Streeter and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your rollator.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

### **Important note**

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

### **Information available**

On our website, <http://www.vermeiren.com/>, you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

## 2 Your product

1. Handgrips
2. Frame
3. Wheels

## 3 Before use

### 3.1 Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- Frame (incl. 4 wheels);
- Instruction manual.

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

### 3.2 Intended use

 **CAUTION** Risk of falling

**The Streeter is meant as support of the leg function during walking. Do not use the rollator if you have other physical limitations that may cause unsafe use of your Streeter, like a weak arm, balance disorders, ...**

The Streeter is designed and produced solely to be used as walking aid for one person with a maximum weight as specified in §7.

The Streeter will compensate the lack of adequate support by the legs by leaning on, or moving with, the Streeter with usage of the upper limbs or chest. The Streeter will provide better stability, support and safety during walking.

The Streeter is intended for indoor use.

It is not intended for transport of goods and/or more people than the user.

It should only be used on surfaces where all wheels are touching the ground. Do not use the Streeter on rocky, bumpy or sandy terrain, or places with a steep inclination. The Streeter may become unstable. Look for flat, stable areas, and bevelled kerbs to walk with your Streeter.

Please check all technical details and limits of your Streeter in §7.

The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

### 3.3 Safety instructions

CAUTION

#### Risk of injuries

**Please read and follow the instructions in this manual. Otherwise you may get injured, or your Streeter may get damaged.**

- Check that no stones or other objects block the wheels of the Streeter.
- While walking, keep both hands on the handgrips.
- Be aware that some parts of your Streeter may get very hot or cold due to ambient temperature, solar radiation, heating devices, etc. Be careful when touching.
- Do not hold the handgrips with wet hands. If you do so, you can lose your grip and lose your balance.
- Do not proceed with your Streeter into traffic. Always stay on the pavement.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing / care.

### 3.4 Symbols on the Streeter

- i** Identification plate is located on the left rear leg at the inner side of the base frame

Maximum weight of user in kg

Type designation

## 3.5 Shipping and storage

The shipping and storage of the Streeter should happen according to the technical specifications in §7. Make sure that the Streeter is stored dry.

Provide sufficient covering or packaging to protect the Streeter from rust and foreign bodies (e.g. salt water, sea air, sand, dust).

Make sure no objects are stored on or against the Streeter to prevent damage.

## 4 Assembly and adjustments

1. Press the spring button until the leg is unlocked.
2. Adjust the leg to the desired length and check that the spring button is secured again into place.
3. Make sure that all legs are adjusted to the same height.

## 5 Using your Streeter

### 5.1 Unfolding / folding

CAUTION

#### Risk of entrapment

**Keep fingers, buckles, jewellery and clothes away from the folding mechanism.**

CAUTION

#### Risk of falling

**Check that the Streeter is completely open before use and that the frame is locked.**

## 6 Maintenance

**i** Regular care ensures that your Streeter is preserved in a perfectly functional condition. For the maintenance manual, refer to the Vermeiren website: [www.vermeiren.com](http://www.vermeiren.com).

### 6.1 Cleaning

Use a soft, damp cloth and lukewarm water with a mild soap to clean the Streeter. Avoid soaking. Do not use abrasive cleaning agents to clean.

### 6.2 Disinfection

CAUTION

**Risk of damage**

**Disinfections may only be undertaken by trained persons. Consult your specialist dealer.**

### 6.3 Inspection / Service / Repairs

The lifespan of the Streeter is influenced by its use, storage, maintenance, service and cleaning.

Before you resume use of the Streeter, it should be inspected on the following points:

- Completeness.
- Folding mechanism: functioning well.
- Bolts and nuts: tightly fastened.
- Wheels (stability, tracking, fixing screws). In case of regular contact with wires, hair or other residues, the wheels may get blocked. Check regularly that the wheels rotate freely and that they are in good condition.
- Frame (deformation, stability, connections).

The service frequency depends on the frequency and intensity of use. Consult your specialist dealer to make an agreement on service frequency.

Repairs and replacements may only be undertaken by trained persons, and only genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

### 6.4 Reuse

Before each reuse, have the Streeter disinfected, inspected and serviced according to the instructions in §6.2 and §6.3 .

### 6.5 Disposal at end of life

At end of life, you need to dispose your Streeter according to the local environmental legislation. The best way to do so, is to disassemble the Streeter to facilitate the transport of recyclable parts.

## 7 Technical specifications

The technical details below are only valid for this Streeter, at standard settings and optimal ambient conditions. Take these details into account during use.

Brand	Vermeiren
Type	Rollator 
Model	Streeter

Description	Dimensions			
	S	M	L	XL
Maximum user weight	50 kg	65 kg	80 kg	90 kg
Overall length	740	900	960	1020
Folded length	200	170	210	210
Overall width	530	550	590	630
Overall height	460-570	470-600	600-730	720-900
Waist	340	360	400	430
Total mass	4,75 kg	5 kg	5,75 kg	6,45 kg
Wheel diameter	Front: 125 mm Rear: 75 mm			
Storage temperature	+5°C - +45°C			
Storage humidity	20% - 75%			
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.				

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:

Vermeiren GROUP  
Vermeirenplein 1 / 15  
2920 Kalmthout  
BE

website: [www.vermeiren.com](http://www.vermeiren.com)

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### All rights reserved, including translation.