VERMEIREN

Four-Light

INSTRUCTION MANUAL
MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

EN

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INSTRUCTION MANUAL 4-wheels rollator Four-Light

facility. A list of Vermeiren facilities can be found on the last page.

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Make	Vermeiren		
Address	Vermeirenplein 1/15, B-2920 Kalmthout		
Type	Rollator		
Model	Four-Light		
Width	660 mm		
Width (folded)	300 mm		
Length (unfolded)	750 mm		
Maximum height	990 mm		
Minimum height	870 mm		
Seathight	550 mm		
Seatwidth	450 mm		
Seatdepth	250 mm		
Weight	8 kg	with basket/ safety belt	8,45 kg
Front wheels	8 x 1 ¼ (Ø 200 mm x 35 mm)		
Rear wheels	Ø 175 mm x 35 mm		
Brakes	Multifunctional brake		
Max. inclination forwards	15°		
Max. inclination backwards and sideways	10°		
Maximum user weight	max. 130 kg		
Maximum basket weight	max. 5 kg		
Turning width	1760 mm		
Storage temperature	+5°C - +41°C		
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / °.			

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.


Before using the rollator, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your rollator. Please remember that observance of our hints would keep your rollator in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren

2. Components

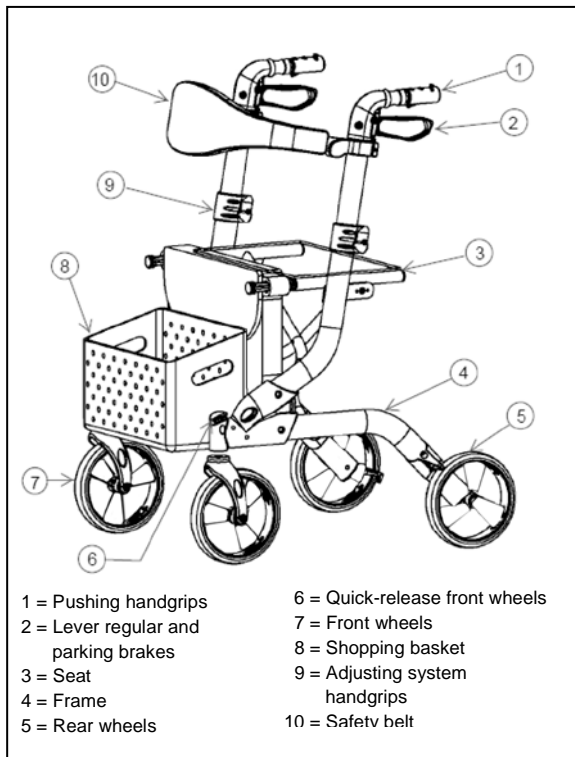3. Explanation of symbols

- Maximum mass
- CE conformity
- Type designation

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- rollator frame with seat, 4 wheels (mounted)
- height adjustable pushing handgrips with brakes
- shopping basket
- safety belt
- manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

5. Intended use

⚠ WARNING: Risk of falling - Do not use the rollator if you have weak arm function or poor equilibrium.

The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, the rollator with usage of the upper limbs or chest. The rollator will provide a better stability, support and safety during walking.

However, it cannot be used by persons with a visible weakened hand and/or arm function and also not by persons with a balance problem.

The rollator can be used indoor and outdoor. The rollator should only be used on surfaces where all four wheels are touching the ground. Not to be used on rocky terrain or steep hills (see technical details: max. inclination) so the rollator can become unstable.

The product is not meant for transport of people and/or goods. The seat can only be used when the parking brake is locked.

6. Assembly

⚠ CAUTION: Chance of pinching - Do not place fingers between the components of the rollator.

⚠ CAUTION: Risk of clamping - Keep fingers away from moving parts of the rollator.

1. Mount the rear wheels

NOTE: The brake cable is attached to the rear wheel. Do not cross the brake cables.

- Mount the rear wheel in the frame of the rollator (rear side). Be aware for the brake cables.
- Push the rear wheel in the tube until the spring button clicks in position.
- Check the rear wheel is firmly secured.
- Repeat for the other rear wheel.

2. Mount the front wheels

- Mount the front wheel axle in the frame of the rollator (front side).
- Push the front wheel in the frame until the front wheel clicks in position.
- Check the front wheel is firmly secured.
- Repeat for the other front wheel.

3. Unfolding rollator

- Remove the attachment plate ① from the tube of the seat.
- Unfold the rollator by pushing on the handgrips.
- Push on the tubes of the seat to unfold the rollator further.
- Check the seat is firmly secured.

4. Adjust the pushing handgrips to the desired height (see further).

5. Cane holder

- Attach the cane holder to the side of the frame by sliding it down over the receptacle located on the side of the handgrip height adjustment button.

7. Dismantling

⚠ **CAUTION: Chance of pinching - Do not place fingers between the components of the rollator.**

⚠ **CAUTION: Risk of clamping - Keep fingers away from moving parts of the rollator.**


1. Adjust the pushing handgrips to the minimum position (see further).

2. Folding rollator

- Pull the seat handgrip ① upwards.
- Push the seat upwards until the rollator is completely folded.
- Click the attachment plate ② to the tube of the seat.

3. Removing front wheels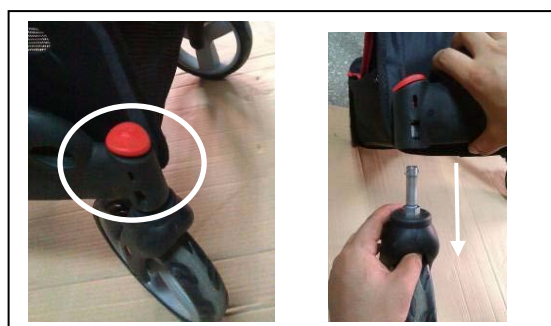

- Press the red button on the front wheel to remove the front wheel.
- Pull the front wheel out the rollator frame.
- Repeat for the other front wheel.

4. Removing rear wheels

- Press the spring button on both sides to remove the rear wheel.
- Remove the rear wheel out the rollator frame.
- Do not remove the brake cables.
- Repeat for the other rear wheel.

8. Pushing handgrips

⚠ **CAUTION: Risk of falling - Firmly fix the handgrips before using the rollator.**

⚠ **CAUTION: Risk of falling - Do not exceed the maximum adjustment, your rollator will not be stable anymore.**

The pushing handgrips can be adjusted in height by following steps:

- Push on the red adjustment button.
- Move the handgrips upwards/downwards until the desired position (6 different positions in steps of 25 mm).

- Loosen the red adjustment button.
- Pull the handgrips a little bit upwards and check they are firmly secured.

9. Safety belt

The safety belt can be adjusted in height by following steps:

- Loosen the star knob to adjust the height of the safety belt.

Mounting /
removing safety
belt:

- Loosen the star knob to remove the safety belt.

10. Brake system


⚠ WARNING: Good operation of the brakes is influenced by wear and contamination of the wheels (water, oil, mud, ...) - Check the condition of the wheels before each use.

⚠ WARNING: The brakes can wear - Check the operation of the brakes before each use.

The rollator has been equipped with a multifunctional brake system. The brake handles underneath the pushing handles are regular brakes as well as parking brakes.

For the usage of a short brake (regular brakes), pull the brake handle upwards (toward the pushing handle).

For the usage of a long brake (parking brake), push the brake handle downwards till you hear a locking sound of the brake. The wheels (use both parking brakes left and right simultaneously) are now blocked until you pull the parking brakes upwards again.

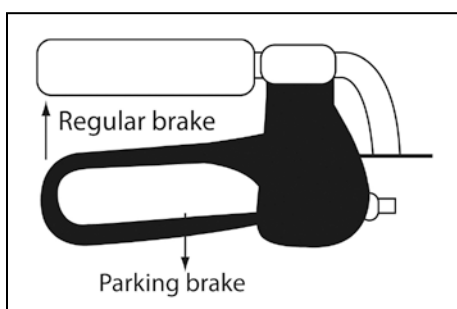11. Basket

⚠ WARNING: Risk of falling, tipping over - Do not overload the shopping basket.

The maximum carrying weight from the shopping basket is 5 kg.

12. Safety instructions

- Ensure there are no stones or any other objects blocking the regular usage of the wheels of the rollator.

- Ensure the brakes or the rollator are always in parking-mode before using your rollator. When you are standing comfortable and stable, you may unlock the parking brakes.
- Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching.
- Before every usage, check if the pushing handgrips are firmly secured.
- Verify if the safety against unwanted folding/unfolding is functioning properly.
- Only proceed moving with your rollator when you are standing comfortably and when both your hands are resting on the pushing handgrips.
- Do not hold the pushing handgrips with wet hands. If you do so, you can lose your grip and lose your balance.
- When you detect that your body is getting tired, lock the parking brakes of your rollator and have a seat on the bench.
- **The maximum users weight for your rollator is 130 kg** and can only be used on a flat underground. Do not use your rollator on a staircase or bumps higher than 40 mm.
- When using the rollator outdoors, find yourself flat and stable areas to use your rollator. Avoid curbs and/or deep holes. Find a place where the curbs are beveled.
- Do not proceed with your rollator into traffic. Always stay on the pavement.
- Avoid bumpy or sandy roads. Otherwise you might lose control over the stability of your rollator.
- Avoid roads with a very steep inclination.
- Do not use your rollator to transport people or items.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

13. Care

It is recommended to use a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Do not use a cleaning agent which contains solvents. Observe the instructions of the cleaning liquid used. Use a damp cloth and lukewarm water for cleaning. Avoid soaking. Do not use metal brushes or any other sharp-edged or pointed cleaning utensils. In the case of damage to the varnish or the plastic work, consult your specialist dealer. Regular care would ensure

that your rollator is preserved in a perfectly functional condition.

14. Disinfection

For sterilisation use only disinfectants suitable for treating varnished metal and plastic surfaces. Observe the instructions of the cleaning liquid used. Use only proved processes (for scrubbing disinfection) and disinfectants included in the list of the Robert Koch Institute (information can be obtained from: www.rki.de). Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

15. Repairs / Service / Inspection

Lifetime of the rollator is influenced by its use, storage, regular maintenance, servicing and cleaning.

You should use only genuine VERMEIREN replacement parts. Repairs and renovations may only be undertaken by trained persons. Make use of the services offered by your specialist dealer. He would gladly assist you further in regard to service and repairs.

Before you resume use of the rollator, it should be serviced according to the following criteria:

- Completeness
- Folding mechanism
- Check the castors (stability, tracking, fixing screws)
- Frames (deformation, stability, connections)
- Pushing handgrips (safe locking from height adjustment system, good support from the handgrips)
- Brakes
- Seat, safety belt and shopping bag on wear or damage
- Cleaning
- **Damaged varnish (remove possible rust and repaint)**

Every 8 weeks:

- Lubricating (moving parts shall be greased)

Every 6 months or for every new user, it should be serviced according to the following criteria:

- General review
- Disinfection

16. Shipping and storage

The shipping and storage of the rollator shall be according following instructions:

- Store in a dry place (between +5°C and +41°C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the rollator from rust and foreign bodies (e.g. salt water, sea air, sand, dust).
- The rollator must be stored without being subjected to strains. (Do not put too heavy parts on the rollator, not clamping between something, ...).

17. Warranty

Excerpt from the "General Business Conditions".

(...)

5. The guarantee period for warranty claims is 24 months.

(...)

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

18. Disposal

When disposing of the rollator, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

19. Declaration of conformity

This product is certified for CE and is classified as a Medical product, Class I.

The CE mark indicates the conformity with standards for protection of health, safety and the environment for products traded within the European economic area.

A copy of the Declaration of conformity is available via our website:

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All rights reserved, including translation.