


Material			
No.	Plan	Article	Quantity
1	P00049610	2670165	1
2	P00050248	2670465	1
3	XP02231	2993879	1
4	XP02229(R)	2993877	1
	XP02230(L)	2993878	1
5	XP02079	GP15300	2
6*	XP02078	GP15226KTL	2
7	XP01914(L)	GP13300	1
	XP01915(R)	GP13400	1
8	XP02922	GP22126KTL	2
9	---	1007521	1
10	M3 DIN985	2994899	2
11	M6x40 ISO7380	2994066	2
12	M6 DIN125	2994414	2
13	P00049630	2997557	1
14	M6	2997556	2

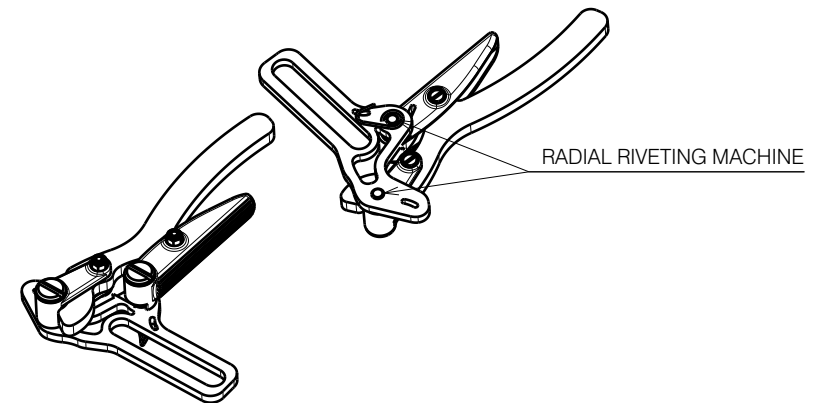

USE WEICONLOCK AN302-43

* Use an radial riveting machine after adjusting and tightening.

SPORTS BRAKE (R,L) V500 ACTIVE				--
				Plan No.
				E00050162/B1.1
Scale:	1:1	Drafted	Verified	Confirmed
Date	30/12/2022	19/06/2023	19/06/2023	
Initials	AS	DR	AS	

