


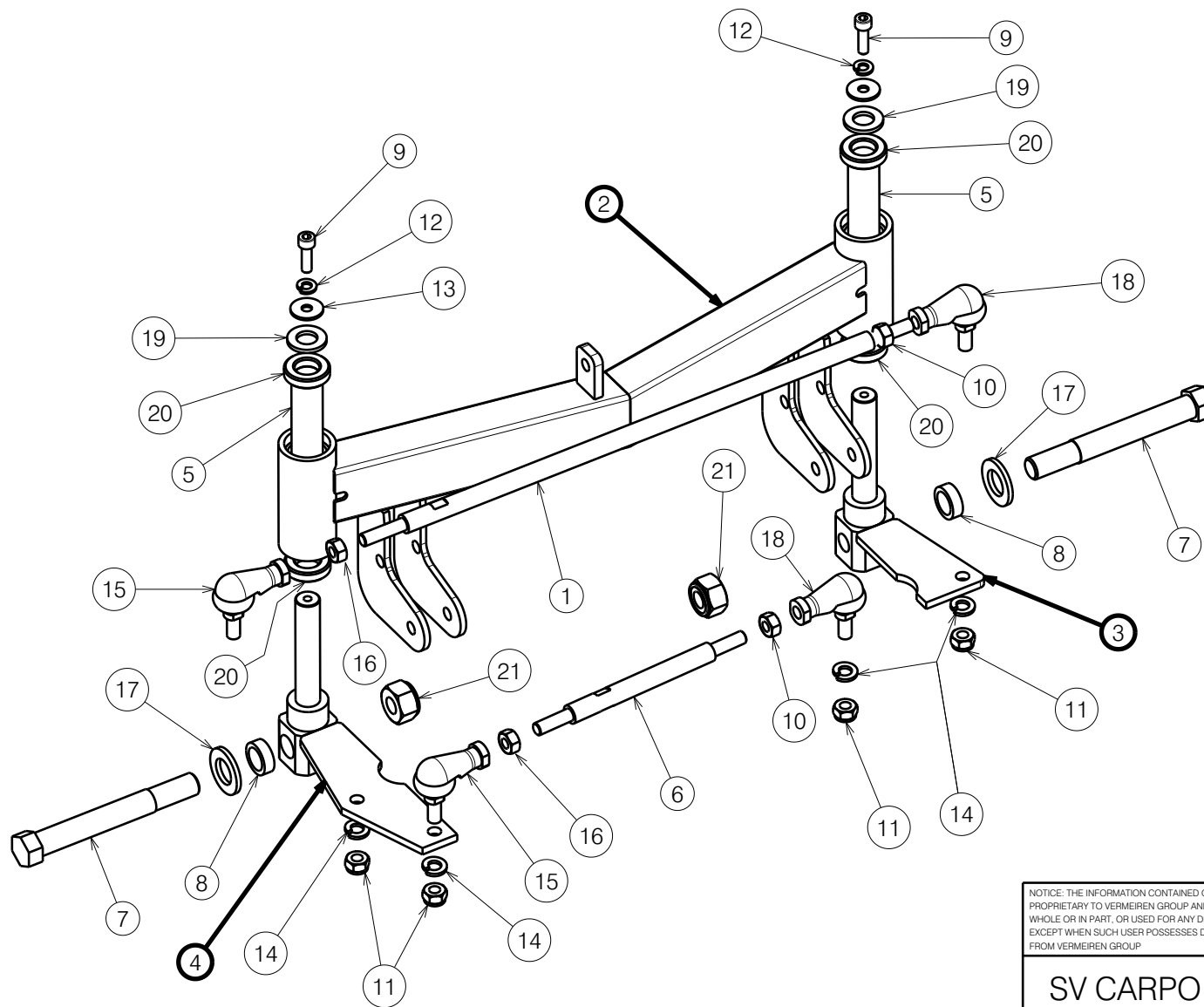

Replaces EC00110

No.	Plan	Article	Qty
1	P00078076	GP30200	1
2	P00076924DP	2685604	1
3	P00076923	2685500	1
4	P00076922	2685400	1
5	P00079125	GP30300	2
6	P00077727	GP30400	1
7	XC00295	3990735	2
8	XC00198	3990729	2
9	M6x20	1001493	2
10	Nut M8	1001671	2
11	Nut M8	1001682	4
12	Washer M6	1004414	2
13	1997328	1997328	2
14	Washer M8	1001570	4
15	Joint left	1922806	2
16	Nut M8 Left	3992037	2
17	Washer M16	1001560	2
18	Joint right	1922805	2
19	1993362	1993362	2
20	Bearing	3991619	4
21	Nut M14	3992772	2

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

SV CARPO 2 SUPPORT FRONT WHEEL ASM

Scale: 1:5	Drafted	Verified	Confirmed
Date	06/03/2024	2/07/2026	2/07/2026
Initials	MK	PM	MK

Plan No.

E00076921/D

