


Material			
No.	Plan	Article	Quantity
1	YP06157	27918...	1
2	YP06164	27944...	1
3	YP06167	27946...	1
4*	YP06196	27950...	4
5	YP06216	2784803	2
6	YP06217	2795303	2
7	XP00106	2990072	2
8	P00097039	2998584	8
9	Wheel D75	1001255	2
10	M6 DIN 985	1001680	4
11	M10 DIN 985	1001683	4
12	---	---	---
13	3.9x16 DIN7982	1005767	4
14	M6x20	1006999	4
15	Velcro 12cm	1990663	0,5m
16	Wheel D75 with brake	1995807	2
17	M10x55	1998518	4
18	Velcro Strap 30cm	2993635	4

* - only for stacking beds / tylko do pietrowania lozek

Velcro straps (No.18) - way of assembly for YP06157 and YP06167 /
Paski Velcro (nr. 18) - sposob montazu dla YP06157 i YP06167

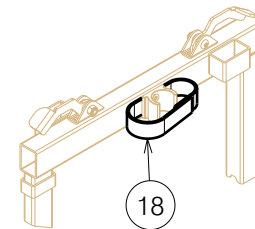


Package with bed dimensions [cm]		
Bed siderails	standard	965
Length	105	105
Width	44	47
Height	140	146
Carton box	U000369	U000367
Stacked beds height	283	289

EP00269 / K


TRANSPORT SYSTEM ILLICO				Plan No.
				1/2 EP00269 / K
Scale: 1:8	Drafted	Verified	Confirmed	
Date	09/01/2018	5/05/2026	5/05/2026	
Initials	DR	PM	AZ	

1

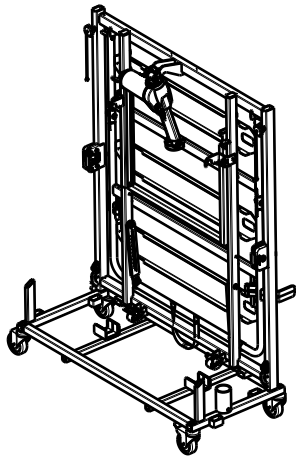


Mount the footside frame /
Zamontuj rame podnozka

2

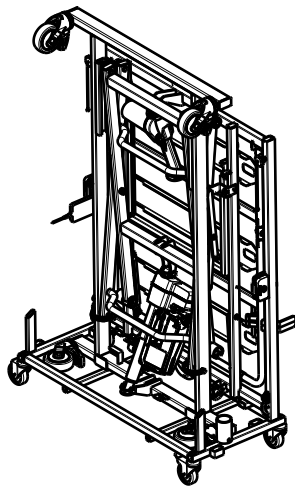


Mount the undercarriage /
Zamontuj rame podnosnika

3

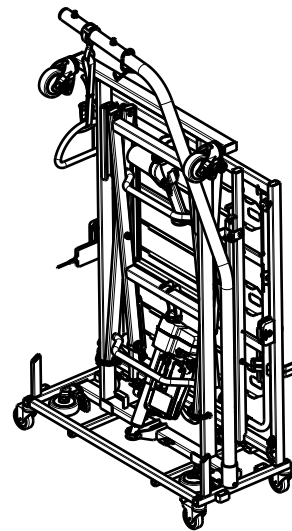


Mount the lifting pole / Zamontuj wysiegnik

4

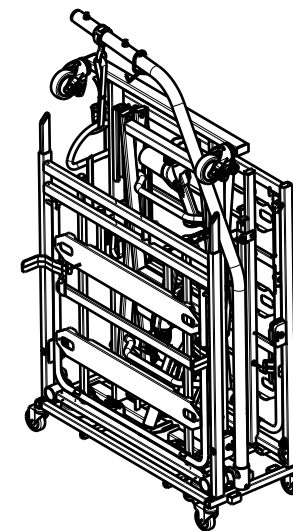


Mount the headside frame /
Zamontuj rame zaglowka

5

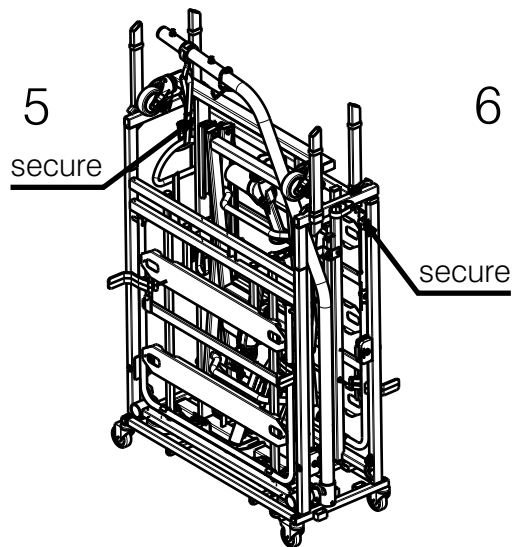


Mount the connectors and secure the connection
using star knob / Zamontuj laczniki systemu trans. i
zabezpiecz polaczenie pokretlem

6

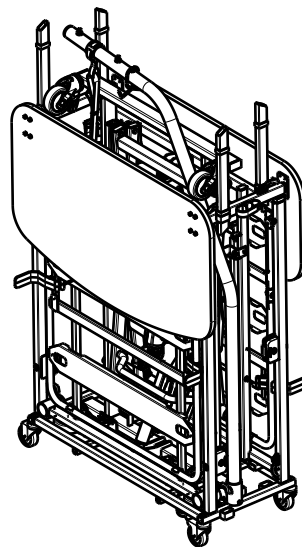


Mount the panels /
Zamontuj panele

7

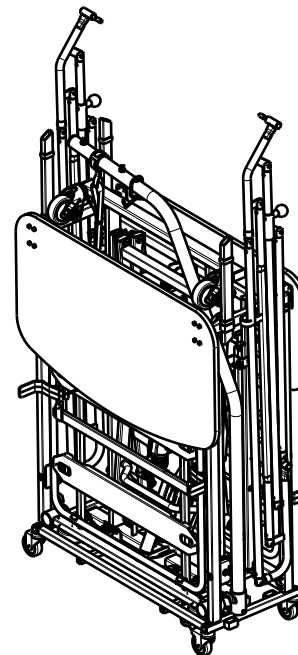

Mount the side rails
/ Zamontuj barierki

8

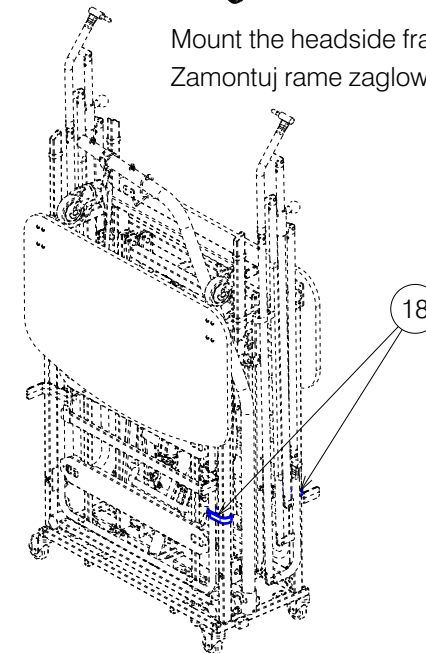

Put on the velcro straps to lock foot and headrest /
zaloz rzepy do blokady podnozka i zaglowka

EP00269 / K

TRANSPORT SYSTEM ILLICO

Plan No.

2/2 EP00269 / K

Scale: 1:8

Drafted

Verified

Confirmed

Date

09/01/2018

5/05/2026

5/05/2026

Initials

DR

PM

AZ

