
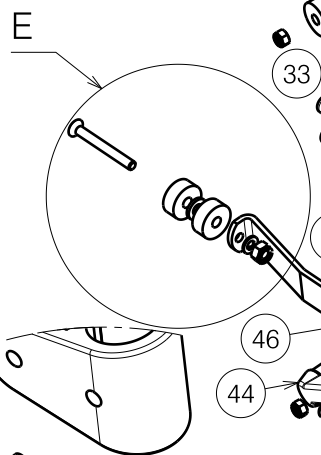
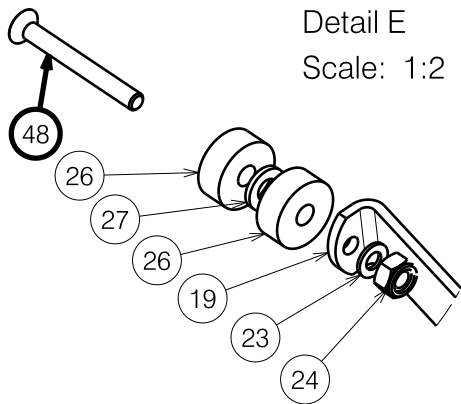





E000184/J

EXPLODED VIEW SEAT FRAME GEMINI 1/3				Plan No.
				E000184/J
Scale : 1:6	Drafted	Verified	Confirmed	
Date	29/04/2005	16/05/2017	16/05/2017	
Initials	BS	DB	TK	

Detail E
Scale: 1:2



Detail D
Scale: 1:2

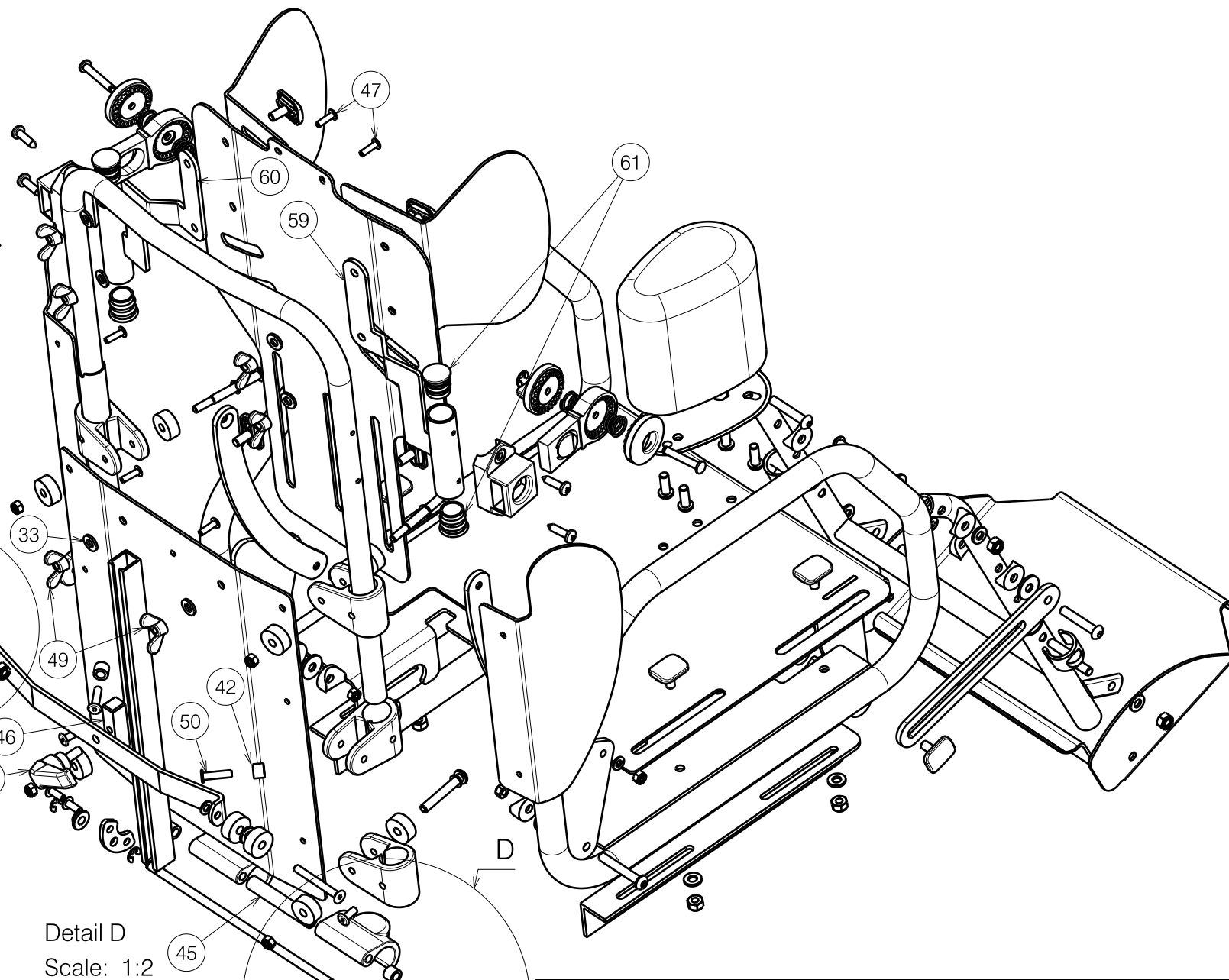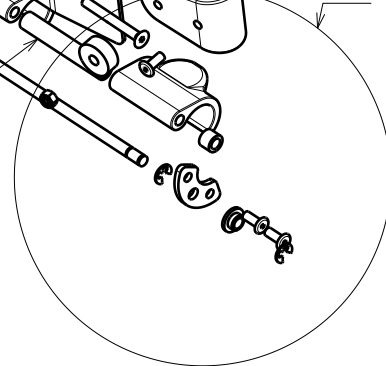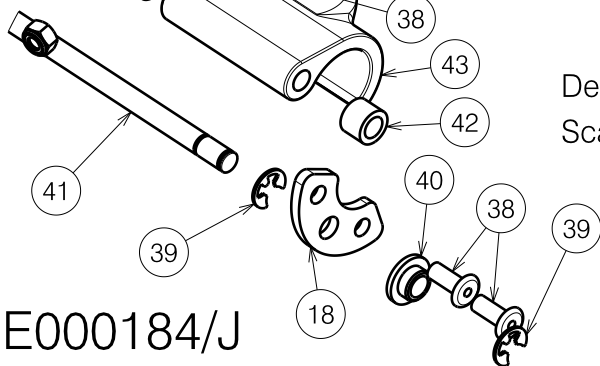

E000184/J

EXPLODED VIEW SEAT FRAME GEMINI 2/3				Plan No. E000184/J
Scale :	1:4	Drafted	Verified	Confirmed
Date	29/04/2005	16/05/2017	16/05/2017	
Init.	BS	DB	TK	



Material			
No.	Plan	Article	Number
1	YP00515	18154..	1
2	YP00500	18151..	1
3	YP00519	18158..	1
4	YP00587	18188..	1
5	YP00586	18157..	1
6	YP00480	18140..	1
7	YP00482	18141..	1
8	(YP00576)	1818482	1
	(YP01182)	1853082	1
9	(YP00575)	1818382	1
	(YP01181)	1852982	1
10	YP00566	18162..	1
11	YP00564	18165..	1
12	YP00511	18153..	2
13	YP00527	18170..	1
14	YP00528	18171..	1
15	YP00501	18236..	2
16	YP00510	18152..	1
17	YP00493	18149..	1
18	(YP00517)	1815600	2
19	YP00485	18143..	1
20	YP00476	18166	1
21	M5x16 DIN7991/8.8	1907493	2
22	D5,3 DIN 9021	1005138	2
23	D5,3 DIN125 A	1001550	4
24	M5 DIN 985	1001679	8
25	M6x40	1006111	2
26	--	2990067	14
27	D6,4 DIN 9021	2990046	4
28	X000586	1005742	8
29	M6 DIN 985	1001680	12
30	--	2990103	2
31	M6x35	1006057	4
32	XP00140	2990097	12
33	D6,4 DIN 125 A	1000973	13

Material			
No.	Plan	Article	Number
34	D6,4 DIN 125 A	1001564	8
35	M6x16	1001564	4
36	M5x35	2990115	6
37	--	2990086	6
38	RIVET 4,8x12	1001585	4
39	--	2990104	2
40	XP00127	1815900	1
41	XP00142	1823800	4
42	X003698	1004550	2
43	--	2990091	1
44	--	1009933	1
45	YP00067	18233	1
46	XP00124	1823700	8
47	RIVET 4x14	1001584	2
48	M5X45 DIN 7991/8.8	2990114	2
49	M6 DIN 315	1001520	2
50	RIVET 5x20	1003661	2
51	X002396	1003675	2
52	YP01036	18926	4
53	--	2990326	4
54	--	2990101	4
55	--	2990102	2
56	--	2990100	2
57	--	2990099	2
58	DIN 6799	1993722	1
59	YP03978	28420	1
60	YP03979	28421	1
61	--	2990095	1
62	XP00817	2991001	1
63	XP04001	28428	1
64	XP00138	18172	2

EXPLODED VIEW SEAT FRAME GEMINI 3/3

Plan No.

E000184/J

Scale :	Drafted	Verified	Confirmed
Date	29/04/2005	16/05/2017	16/05/2017
Initials	BS	DB	TK



E000184/J