
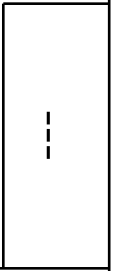


Detail A
Scale: 2:5

E001112 / B


SIGMA BACK FRAME L44 SE41 Sheet 1/2				Plan No. E001112 / B
Scale: 1:5	Drafted	Verified	Confirmed	
Date	23/08/2018	18/03/2019	18/03/2019	
Initials	SD	SD	PJ	



Material			
No.	Plan	Article	Quantity
1	Y022312	2049854 (40)	1
	Y022980	2049754 (45)	1
	Y022313	2049954 (50)	1
2	Y023126	2053954 (40)	1
	Y023127	2054054 (45)	1
	Y023128	2054154 (50)	1
3	X005410	1914781	1
4	M8x16 DIN912	1001498	4
5	X001119	1001554	4
6	M8 DIN125	1001556	2
7	M4 DIN985	1001678	2
8	M8 DIN985	1001682	4
9	M10 DIN985	1001683	1
10	X000862	1006463	4
11	M10x35 ISO7380	1009442	1
12	Y018564	1774200	2
13	M8x50 ISO7380	1902483	2
14	X004888	1906839 (40)	1
	X004857	1906740 (45)	1
	X004889	1906841 (50)	1
15	LA23	1906790	1
16*	Cable	1906791	1
17*	Lock	1906792	1
18	---	1906843 (40)	1
	---	1906844 (45)	1
	---	1906845 (50)	1
19	M4x12 DIN7984	1907582	2
20	M5x10	1911896	1
21	ACT Linx	1912307	1
22	D22	1990648	4
23	D10x40	1991481	1
24	M8x20 ISO7380	1992349	2
25	30x20	1997922	2
26	---	2993352	1
27	∅35	T001002	1

* Not depicted

E001112 / B

SIGMA BACK FRAME L44 SE41				Plan No.
Sheet 2/2				E001112 / B
Scale: 1:5	Drafted	Verified	Confirmed	
Date	23/08/2018	18/03/2019	18/03/2019	
Initials	SD	SD	PJ	

